Freiheitsmittel
Ways of freedom
Instrumentos de libertad
Moyens de liberté

STANDARD
JUNIOR 2
MINI RAMP
RAMP
PLUS 2006

JUNIOR PLUS 2006

Jolly

PATENTIERT

TECHNOLOGIE, DIE HILFT • USEFUL TECHNOLOGY • TECHNOLOGÍA ÚTIL • TECHNOLOGIE QUI AIDE

Jolly

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HILFSMITTEL NR. 18.65.01.2005

 The stairclimber Jolly is a mobile stairclimber with tracks designed for the overcoming of architectural barriers; it does not require any wall-mounted structure, and it is destined for the transport of wheelchairs on stairs.

Due to the great demand, many models and accessories were developed; the model Jolly 2006 Plus successfully meets a wide range of different needs in particular. The attendant operates it only making use of the controls provided.

It can be dismantled in two separate pieces in order to be easily loaded and transported into the car.

All models provided with loading ramps are suitable for the transport of electronic wheelchairs, wheelchairs with tilting or movable backrest and for pushchairs.

- **Free no obligation building assessments, and free training how to operate; training is performed directly at user's home.**
- **After-sale service by T.G.R. authorised centres all over the country.**
- **Please carefully follow all prescriptions, warnings and instructions given in the operation and maintenance handbook provided with the machine.**

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*On request reinforced models for the transport up to 150 Kg.

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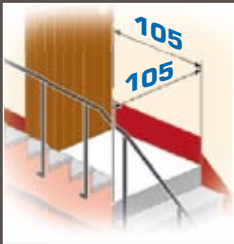
 This version is particularly suitable for the transport of wheelchairs with a narrow distance from the ground and/or a narrow distance between wheels. The loading ramp can be easily removed, thus allowing a fast and comfortable transport into the car.

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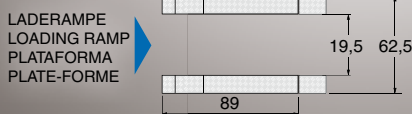
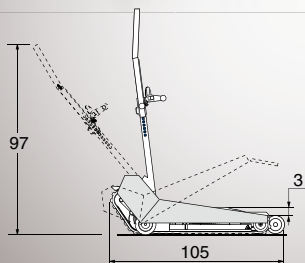
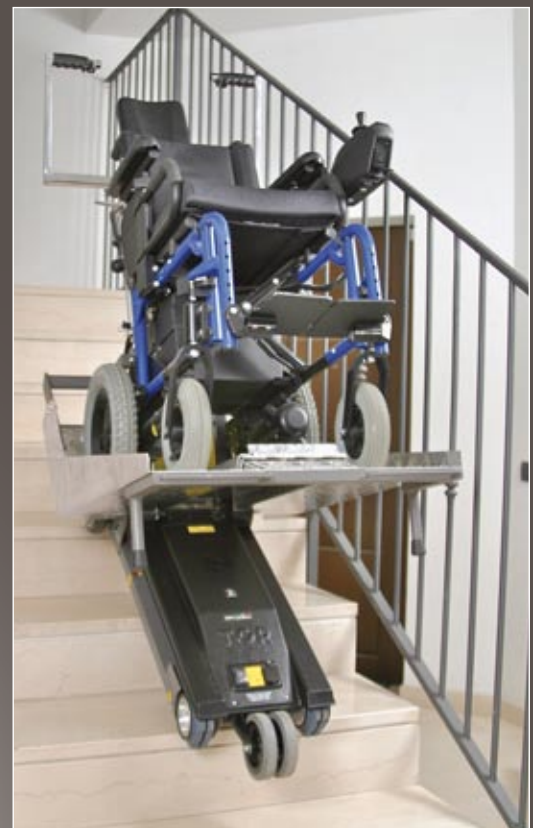
 The Ramp version allows you to load and transport wheelchairs with a bigger distance from the ground and, as a general rule, wheelchairs with bigger overall dimensions and marked backrests. The loading ramp cannot be removed from the machine's frame.

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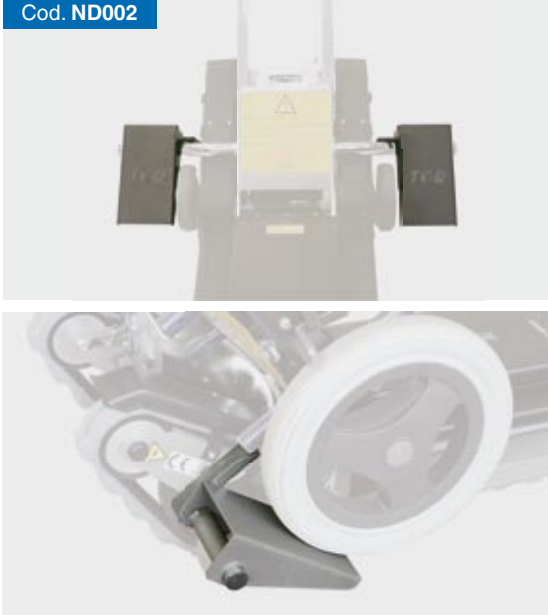
 Designed to make the load and unloading procedure easy and effortless, it has to be used when the wheelchair has already been attached to the steering wheel. It helps to reduce the physical effort.

It can be applied also on the model Junior 2.

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 The seat fits directly to the stairway climber. Designed to be used from those people who, despite some mobility-related difficulties, do not need to use a wheelchair.

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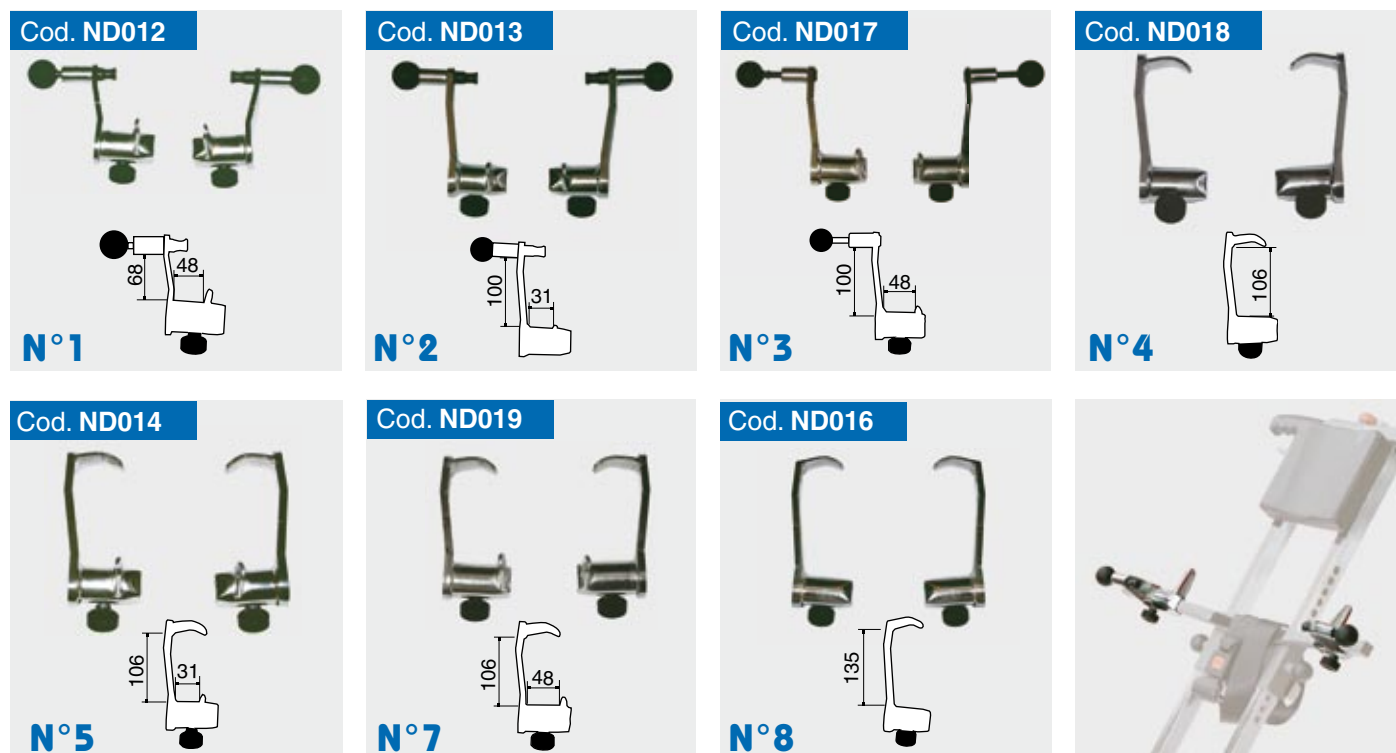
 They can be applied in different positions on **the whole Jolly Plus** range, they make the rear wheels resting properly.

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THE STAIRCLIMBER HAS BEEN EQUIPPED WITH:

- Height- and width- adjustable fasteners which fit most types of wheelchairs (only for standard model)
- Base designed to accommodate even wheelchairs with very low cross brace
- Headrest adjustable in height
- Adjustable safety belt
- Key switch restricting the device use to authorised personnel, only
- **Emergency button**
- Safety device which enables the brakes automatically
- Ascent/descent controls placed on the steering wheel handlebar
- Non-skid rubber tracks that leave no marks
- 2 wheels of small diameter allowing easy manoeuvring on horizontal surfaces
- **Built-in electronic battery charger**
- Two-coloured led indicating the battery recharge status
- **Double security device** which prevents the machine to be started-up when the device has not been properly locked and – at the same time – prevents the steering wheel to detach himself from the machine during operation on stairs
- **Safety device to stop the climber automatically in case an anomaly occurs**
- **Brake release button** that in case of emergency stop on stairs - thanks to the **reversible power** transmission - allows you to release the brake and reach the maximum stability on stairs and/or landing at a low and controlled speed
- Device to signal the battery power-level
- Indicator for checking the stairway slope
- **Ascent range 50 floors with a weight of 130 kg**
- **Overcoming slope up to 35° (equivalent to an incline of 70%)**
- Max. step height 18 cm
- **24Month warranty**

Motor:	12V - 160W, direct current
Speed:	5,7 m/ min
Battery:	12V 24Ah, sealed, rechargeable, without maintenance
Brake:	12V electromagnetic negative
Transmission:	Toothed belt, reducer with steel gears cemented and tempered
Battery charger:	The brake is applied on the reducer, directly built-in, electronic, in steady current

The data are indicative and not binding. T.G.R. reserves the right to change them at any time.

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T.G.R. S.r.l.
T.G.R. GmbH