D

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Rollator XXL and Rollator XXL including Arthritis-Armrests

Instructions for Use

Reutäckerstrasse 12
D-76307 Karlsbad-Ittersbach
Tel. +49 (0) 7248/ 9186 - 0
Fax. +49 (0) 7248/ 9186 – 86
info@dietz-reha.de

Rollator XXL and Rollator XXL including Arthritis-Armrests
Instructions of Use Version 4.0.0
Status 03/2015 _ LW

General

Compliance with EC Directive 93/42 for medical products is confirmed for the Rollator XXL and the Rollator XXL including Arthritis-Armrests (further not specific named) described in the following. These instructions for use are part of the scope of delivery. The user must store these instructions in a safe place and keep them with the product if it is passed on to another user. We reserve the right to alterations due to technical developments compared to the design described in the instructions for use. Reproduction, translations and copies in any form, including extracts, require written approval from the manufacturer. The manufacturer is owner of copyright. These instructions for use are not subject to an amendment service. We will provide you with the current status, on request.

Indication and Purpose of Use

The Rollator XXL serves as a walking support for persons with mobility disorders and max. 200 kg body weight and also as a seat for temporary rest. The wire basket suspended beneath the seat is for storage and transporting items and goods. The Rollator XXL must only be used on solid, even ground. Use for other purposes, especially as a set-up or to transport heavy loads is inappropriate and dangerous!

Setting up / Unfolding the Rollator

- Position the rollator in front of you in the direction of motion.
- Actuate the locking brake.
- Hold the rollator with one hand on the handle.
- With the other hand, press the seat down until you feel it lock in place.
- Fasten the back rest.
- Hang the basket in the frame from the front.


Folding the rollator

- Remove the basket from the frame.
- Position the rollator in front of you in the direction of motion and actuate the locking brake.
- Hold the rollator with one hand.
- With the other hand, firmly remove the seat from its bracing.
- Now pull the rollator together.
- You can dismantle the back rest and the push handles for better storage.

Setting the push handles


- First actuate the locking brake.
- Unscrew the screws and remove the fastening screws.
- Position the push handles at the correct height (recommended: push handles should be at the height of your wrists).
- Replace the fastening screws and tighten the star grip nuts.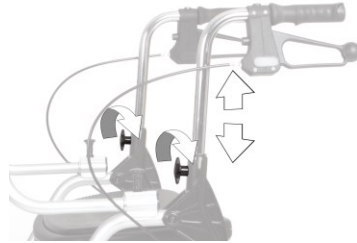

Make sure the screw sinks completely into the drilling hole.
You can remove the push handles completely from the frame for better storage.

Setting the Arthritis-Armrests


For the setting of the Arthritis-Armrests please operate identical as for the setting of the push handles.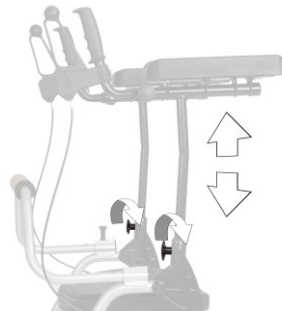

Setting the grip handle distance

The distance of the grip handles can be adjusted stepwise.


- First actuate the locking brake.
- Unscrew the screws and remove the fastening screws.
- Position the grip handles at the position which is comfortable for you.
- Replace the fastening screws and tighten the star grip nuts.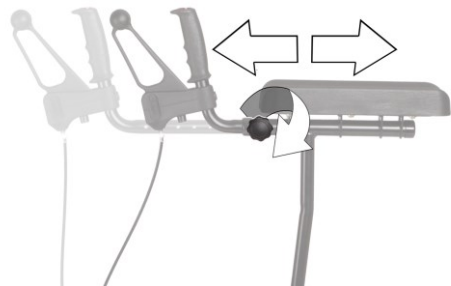

Rollator XXL

Fastening the back rest

- Pull the snap locks upwards with a slight twist so that the lock is released.
- Push the back rest into the drilling holes.
- Now actuate the snap locks so that they lock in completely.
- For dismantling, proceed in reverse order. You can store the back rest separately when storing the rollator.

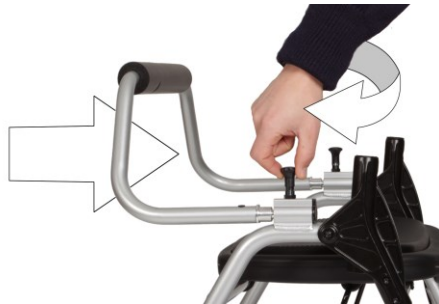Operating and Locking brake

- The brake is used as an operating brake (when pushing) by pulling the brake lever upwards.
- By pushing the lever downwards, the brake is used as a locking brake (for parking)

Please make sure that the operating and locking brake are correct adjusted and are working flawless before using the rollator.

Setting the brakes

- The braking force can be set directly at the brake lever or the Bowden cable.
- The setting of the brakes is correct if the brake lever and Bowden cable have no clearance and the wheels turn without grinding.

Safety Notes

- Do not hang any bags on the handles.
- Use the operating brake when using the rollator on uneven paths.
- Only use the rollator on horizontal, solid ground.
- The rollator may tilt backwards if you lean on it too strongly.
- Clothing or parts of the body may get jammed when folding the Rollator XXL together.
- Please remember the cushion parts subjected to direct sunlight heat up and may cause injury to the skin contact. Cover these parts or protect the rollator from sunlight.

Cleaning

Cleaning and disinfection should be carried out at regular intervals (essential if the user changes). Use mild soap suds without additives for cleaning. Use conventional disinfectants for disinfection. Do not use steam jet units or any scouring or caustic cleaning agents!

Maintenance

The Rollator XXL is maintenance-free. If you notice failing of the brakes, stiffness of the wheels or other functional impairments, the Rollator must be given to a specialist trader for servicing.

Scope of delivery

- Rollator XXL
- Shopping basket
- Back rest
- Instructions for Use

Guarantee

The guarantee lasts 24 months and covers all defects of the product which are verifiably caused by material or manufacturing errors.

Technical Data

Maximum added weight:	200 kg / 3 kg shopping basket
Empty weight:	12.6 kg incl. shopping basket
Total width:	710 mm
Total depth:	730 mm
Total height:	820 – 925 mm
Turning radius:	1360 mm
Distance between handles:	640 mm
Seat height:	540 mm
Adjustments:	Push handle: 820-920 mm (adjustable in 5 steps)
Tires:	Ø 200 x 30 mm, puncture-proof

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