# GIGO

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- 1 Handgrip
- 2 Handle tubing (height adjustable)
- 3 Brake
- 4 Height adjustment screw for handles
- 5 Rear wheel
- 6 Frame
- 7 Front wheel
- 8 Basket
- 9 Seat
- 10 Tray
- 11 Brake wire
- 12 Back Rest
- 13 Cane holder

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Walking device for indoor and outdoor use for compensation and improvement respectively of restricted ability of walking.

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This product is suitable for the reapplication. With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

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## Lieferumfang – Scope of delivery contents – Geleverd product

- |                              |                           |
|------------------------------|---------------------------|
| <b>1. Rollator</b>           | 1. Walking device         |
| <b>2. Korb</b>               | 2. Basket                 |
| <b>3. Tablett</b>            | 3. Tray                   |
| <b>4. Rückenstütze</b>       | 4. backrest               |
| <b>5. Gebrauchsanweisung</b> | 5. Operating instructions |
- 
- |                       |
|-----------------------|
| 1. Rollator           |
| 2. Mand               |
| 3. Dienblad           |
| 4. Rugsteun           |
| 5. Gebruiksaanwijzing |

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Insert the back support from the top into the reception tubes and protect with the Snap ring as illustrated.

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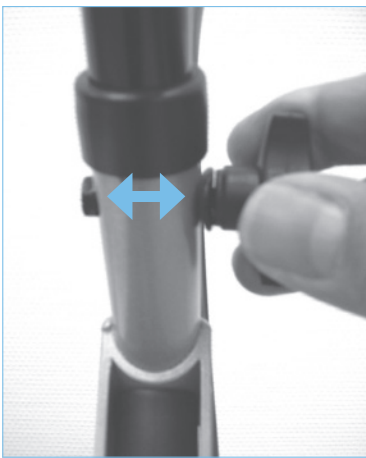
Insert the tube of the adjustable handgrip into the frame at least to the "first hole". Insert the locking screw through the pipe from the inside and fix with washer and star clutch screw.

To change the height setting of the adjustable handgrip the locking screw by turning off the star clutch against the clockwise solve. Then ascertain as on top described the clutch by desired height.

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Hook in the basket with the hooks (see circle) at the frame under the seat plate and place the tray with its small feet in the holes in the seat).

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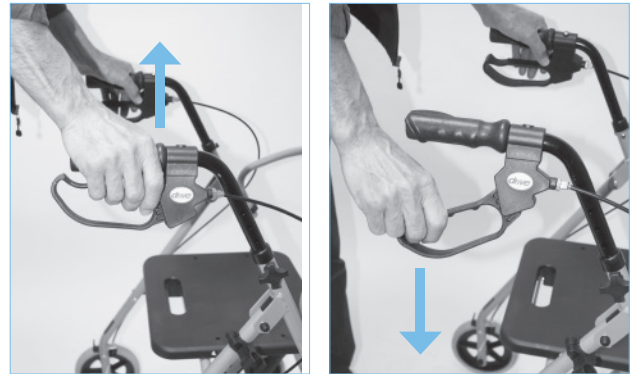
For braking the walking device pull both brakes upwards to the handgrip (see picture). Always brake slowly and steady.

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For parking the walking device press down the brakes until they click in the parking position (see picture). For un-locking the brakes pull them upwards towards the handgrips until they are unlocked (both brakes are in a middle position).

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The effect of the brakes can be affected by abrasion of the wheels. Therefore test regularly the brakes.

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For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 1) or with the suitable tool (not included) on the brake re (picture 2).

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Position the walking device on its wheels. Push the front frame forward by pressing on a handgrip down until the walking device stands on four wheels.

Then press the seat down until it clicks into place.(3) Protect brake wires by shutting iat the pipes (4).

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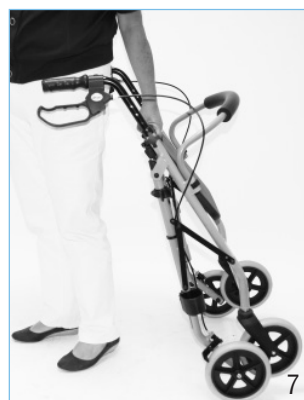
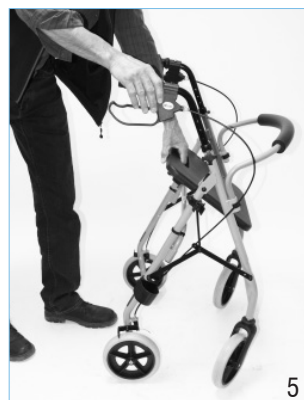
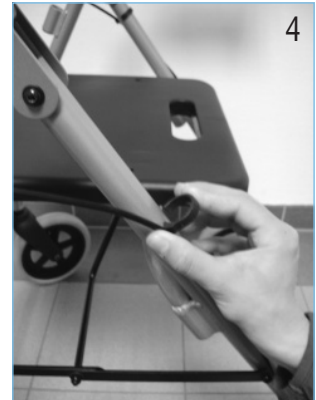
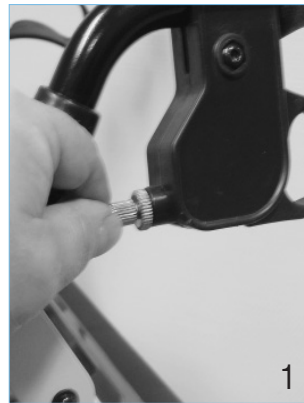
To fold the Rollator, press the red interlock bracket in the Cut-out of the seat plate to the outside and Pull the seat then upwards. Then slant Rollator against the run direction, the frame (picture 5+6) wrinkles. Now the Rollator can be move like a trolley or be stowed away (picture 7).

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- 1 Product name
- 2 Article number
- 3 Serial number
- 4 Date of manufacturing
- 5 Attention: See operating instructions

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Drive Medical GmbH & Co. KG · 88316 Isny/Allgäu (Germany)