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BESTELL  
KATALOG

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
# Sortiment DEHP freier Infusionsbestecke

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
Spike Set PN DEHP free  
KM1.EE.148.0

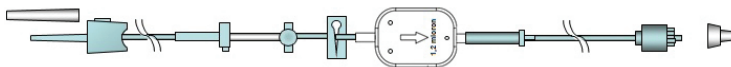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Pediatric Spike PN Set DEHP Free 0,2um  
filter for 2 in 1 F  
KE1.EE.192.9

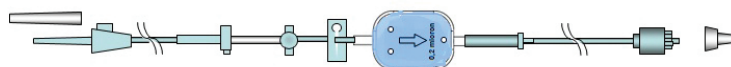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Double Short Spike Connecting line  
with back check valve DEHP Free  
KE1.EE.225.1

Triple Short Spike Connecting line with  
back check valve DEHP Free  
KE1.EE.226.1

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KM1.YY.564.4 – Rucksack Pediatric 2L Pink (4RW)  
KM1.YY.565.4 – Rucksack Pediatric 2L Blue (4RW)  
KM1.YY.578.4 – Rucksack Pediatric 2L Black (4RW)  
KS5.04.197.5 – Rucksack Pink 2L with removable wheels  
KS5.04.198.5 – Rucksack Blue 2L with removable wheels  
KS5.04.200.5 – Rucksack Black 2L with removable wheels

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KM1.YY.481.3 – Rucksack Adult Black 4L (2 pumps)  
KS5.04.207.3 – Rucksack Adult 4L - 2 pumps with removable wheels

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Freedom without clinical compromise

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1. Staun M. et al. ESPEN Guidelines on Parenteral Nutrition: Home Parenteral Nutrition (HPN) in adult patients. *Clinical Nutrition*. 2008;28:467-479. 2. Small M. et al. Using Remote Monitoring to Detect Infusion Related Issues in Patients on Home Parenteral Nutrition. *Clinical Nutrition*. 2015;34:29. 3. Saqui O, Fernandes G, Allard J. Quality of Life Analysis During Transition from Stationary to Portable Infusion Pump in Home Parenteral Nutrition Patients: A Canadian Experience. *Nutrition in Clinical Practice*. 2014;29(1):131-141. 4. Stout S. et al. Metabolic Effects of Cyclic Parenteral Nutrition Infusion in Adults and Children. *Nutrition in Clinical Practice*. 2010;25(3):277-281. 5. Micrel customer perception survey. 31. Mai 2018.

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
## Set-up Guide


The only dedicated portable Home Parenteral Nutrition system to meet all the requirements of both adult and paediatric patients.

## MODE - Enter Code 0112


### Volume Over Time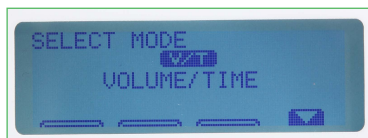


### Intermittent

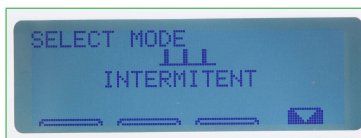
### Auto Ramp
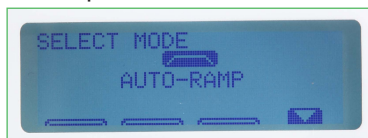
### Continuous
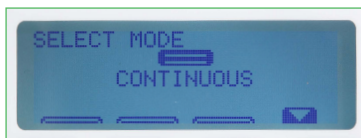
### 25 Steps

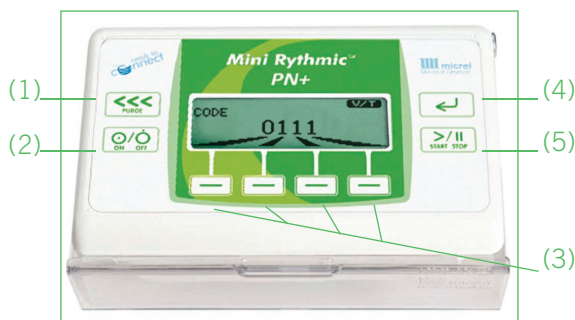- (1) **PURGE BUTTON**  
used to prime non gravity giving set
- (2) **ON/OFF BUTTON**  
one button with dual action
- (3) **SOFT TOUCH BUTTONS**  
with multi-functions
- (4) **ENTER BUTTON**  
used to confirm entries
- (5) **START/STOP BUTTON**  
one button with dual action

Some features mentioned can be disabled in the configuration menu. Please be aware that the pump may have been customised after the factory settings have been installed.

### **Programming the Pump**

- Turn pump on by pressing and holding button (2) **enter code 0111**; press enter button (4)
- Ensure the pump is in the correct mode
- Select required programme

### **New Patient Programme (Volume Over Time)**

- Set volume; press enter button (4)
- Set volume to be infused (VTBI); press enter button (4)
- Set infusion time; press enter button (4)
- Delay time insert time if required or; press enter button (4)
- Screen will now confirm all details entered; press enter button (4)
- Pump on Hold if details are correct; double press start/stop button (5) to start the pump

### **New Patient Programme (Auto – Ramp)**

- Set volume; press enter button (4)
- Set volume to be infused (VTBI); press enter button (4)
- Set taper up time; press enter button (4)
- Set taper down time; press enter button (4)
- Set infusion time; press enter button (4)
- Delay time insert time if required or; press enter button (4)
- Screen will now confirm all details entered; press enter button (4)
- Pump on Hold if all details correct; double press start/stop button (5) to start the pump

### **New Patient Programme (Continuous)**

- Set required rate; press enter button (4)
- Set volume; press enter button (4)
- Screen will now confirm all details entered; press enter button (4)
- Pump on Hold if all details correct; double press start/stop button (5) to start the pump

### **New Patient Programme (25 steps)**

- Set bag volume; press enter button (4)
- Set number of periods; press enter button (4)
- Set infusion time for each step; press enter button (4)
- Set total volume for each step; press enter button (4)
- Delay time insert time if required or; press enter button (4)
- Screen will now confirm all details entered; press enter button (4)
- Pump on Hold if all details correct; double press start/stop button (5) to start the pump

Intermittent modes allows for dose cycle of infusions associated with HPN.

### **New Bag**

Confirmation screen will appear with the information from the previous bag, if this is correct; press the enter button (4).

- Delay time insert time if required or; press enter button (4)
- Pump on Hold; double press start/stop button (5) to start pump

### **Resume after Power Off**

- Pump on Hold double; press start/stop button (5) to restart pump

## PRIMING THE GIVING SET

### Manual Priming

1. Spike the bag – keep filter vertical any air will be eliminated via the filter.
2. Press priming button (do not use a sharp object) until filter is saturated.
3. Turn the priming button upside down, continue to press until the giving set is fully primed.

### Pump Priming

Close clamp prior to spiking the bag. Load giving set into pump and follow pump programming until pump on hold. Press purge button (1) to commence priming. When giving set is fully primed; press purge button (1) to stop priming.

### Pump Placement

Placing the pump in the rucksack – instructions are provided in every new rucksack.

Placing the pump in the pole clamp.


## WARNINGS

Pump will continue to function. Alarm will sound intermittently.

- Press soft key below the bell sign to silence the alarm

### **Near End of Infusion – N.E.O.I**

Pump will alarm when nearing the end of the VTBI. Time is preset at 20 minutes. This can be changed in the configuration menu to volume remaining or time remaining, or turned off.

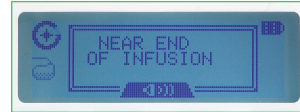### **Low External Battery Pack (EBP)**

Alarm will sound 40 minutes before the battery is completely depleted. The Internal battery will AUTOMATICALLY take over the running of the pump.

- Press the bell sign twice to silence the alarm
- Change to another EBP or connect to the mains supply

### **Low Internal Battery**

Alarm will sound 15 minutes before internal battery is depleted.

- Press bell sign twice to silence the alarm
- Connect to EBP or mains supply and change internal battery when available
- Ensure internal battery indicator is showing 2 or more blocks

### **Keep Vein Open (KVO )**

Alarm will sound when pump goes into KVO mode.

- Press enter button (4) to silence the alarm. Pump will stop and an alarm sound when VTBI has been reached
- KVO will only function if the bag volume is greater than the VTBI

### **Pump Unattended**

Intermittent alarm will sound when the pump is on hold for an hour and no button has been pressed or pump started. This will last for 1 hour.

- Press enter button (4) confirm settings and start infusion

## ALARMS

When there is an immediate problem the pump will Alarm and stop functioning until the problem has been resolved.

### Downstream occlusion

Indicates there is a problem between the pump and the patient.

- Press enter button (4) to silence the alarm  
Rectify the problem and pump will automatically restart

### Upstream occlusion or air in line

Indicates there is a problem between the bag and the pump.

- Rectify the problem; press enter button (4) to silence the alarm  
then restart pump by; press stop/start button (5)

### Depleted Internal Battery


Indicates the internal battery requires renewing.

- Press enter button (4) to silence the alarm. Turn the pump off using the stop/start button (4) and replace the Internal battery
- Turn the pump on using the on/off button (1) and restart pump by selecting resume after power off programme and pressing stop/start button (5)

### End of Infusion

VTBI has been reached and/or KVO is complete.


- Press start/stop button (5) to turn pump off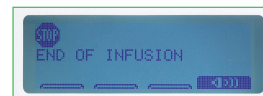

### Door open

This indicates either the internal door or external casing is not closed properly. This alarm will also appear if an attempt has been made to start the pump with no giving set in place.

- Press enter button (4) to silence the alarm
- Ensure doors are closed and giving set is loaded
- Press start/stop button (5) twice to start the pump

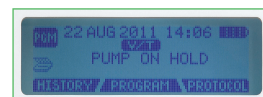Name of Clinical Contact \_\_\_\_\_

Phone number \_\_\_\_\_

Email address \_\_\_\_\_

We support healthcare providers by following a simple philosophy of 'Patient First' and aim to provide absolute excellence in customer service with a fantastic product range supported by a range of technical service programmes.

**Inspiration Healthcare Ltd**

Gildor House, West Street, Earl Shilton, Leicester, LE9 7EJ, UK

**Tel:** +44 (0) 1455 840 555 **Fax:** +44 (0) 1455 841 464

**Email:** [info@inspiration-healthcare.com](mailto:info@inspiration-healthcare.com) **Web:** [www.inspiration-healthcare.com](http://www.inspiration-healthcare.com)

**24 hour emergency hire and support call** +44 (0) 1455 840 555

500-402(4)