
Turner

Manual

etac®

2013-08-26 74274M

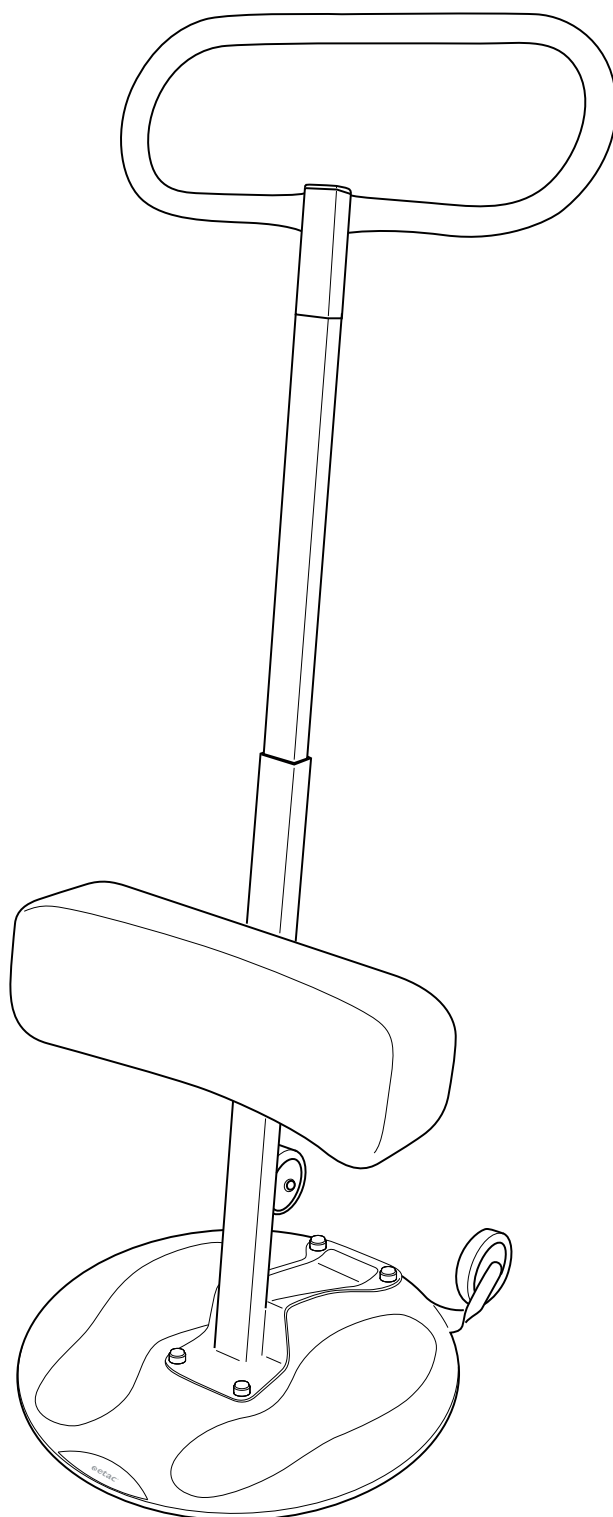


Innehåll / Innhold / Indhold / Sisälllys / Contents / Table des matières / Inhalt / Inhoud / Contenido / Contenuto

SE Teknisk information	3	FR Informations techniques	3
Tillbehör	3	Accessoires	3
Montering	4	Montage	4
Avsedd användning	4	Usage	5
Handhavande	6	Mode d'emploi	6
Säkerhet , garanti	8	Sécurité, garantie	8
Tvätt	9	Nettoyage	9
Underhåll	10	Entretien	10
Rekonditioneringsanvisning	11	Instructions de reconditionnement	11
NO Teknisk informasjon	3	DE Technische Informationen	3
Tilbehør	3	Zubehör	3
Montering	4	Montage	4
Tiltenkt bruk	4	Verwendungszweck	5
Håndtering	6	Anleitung	7
Sikkerhet, garanti	8	Sicherheit, Garantie	8
Vask	9	Reinigung und Desinfektion	9
Vedlikehold	10	Wartung	10
Rekondisjoneringsanvisninger	11	Instandsetzungsanleitung	11
DA Teknisk information	3	NL Technische gegevens	3
Tilbehør	3	Accessoires	3
Montering	4	Montage	4
Anvendelse	4	Beoogd gebruik	5
Håndtering	6	Zo werkt het	7
Sikkerhed, garanti	8	Veiligheid, garantie	8
Vask	9	Wassen	9
Vedligeholdelse	10	Onderhoud	11
Renoveringsanvisninger	11	Instructies voor revisie	11
FI Tekniset tiedot	3	ES Datos técnicos	3
Tarvikkeet	3	Accesorios	3
Kokoaminen	4	Montaje	4
Käyttötarkoitus	4	Uso previsto	5
Toiminta	6	Utilización	7
Turvallisuus, takuu	8	Seguridad, garantía	9
Pesu	9	Lavado	9
Ylläpito	10	Mantenimiento	11
Kunnostusohje	11	Instrucciones de ajuste	11
EN Technical information	3	IT Dati tecnici	3
Accessories	3	Accessori	3
Mounting	4	Montaggio	4
Intended use	5	Uso previsto	5
How to use	6	Istruzioni per l'uso	7
Safety, warranty	8	Sicurezza, garanzia	9
Washing	9	Lavaggio	9
Maintenance	10	Manutenzione	11
Reconditioning instruction	11	Istruzioni di riparazione	11

Teknisk informasjon / Teknisk informasjon / Teknisk informasjon / Teknisk informasjon / Technical information / Informations techniques / Technische Informationen / Technische gegevens / Datos técnicos / Dati tecnici

- SE** Max last 150 kg
Totalvikt 12,8 kg
Material: Pulverlackerat stål
- NO** Maks last 150 kg
Totalvekt 12,8 kg
Materiale: Pulverlakert stål
- DA** Maks last 150 kg
Vægt 12,8 kg
Materiale: Pulverlakeret stål
- FI** Enimmäiskuormitettavuus 150 kg
Kokonaispaino 12,8 kg
Materiaali: pulverimaalattu teräs
- EN** Max load 150 kg
Weight 12,8 kg
Material: Powder-coated steel
- FR** Charge max 150 kg
Poids total 12,8 kg
Matériau: Acier peint par poudrage
- DE** Max. Tragfähigkeit 150 kg
Gewicht 12,8 kg
Material: Stahl, pulverbeschichtet
- NL** Veilige werkbelasting 150 kg
Gewicht 12,8 kg
Materiaal: Staal
- ES** Carga de Trabajo Segura 150 kg
Peso total 12,8 kg
Material: Acero
- IT** Safe Working Load 150 kg
Peso complessivo 12,8 kg
Materiali: Acciaio

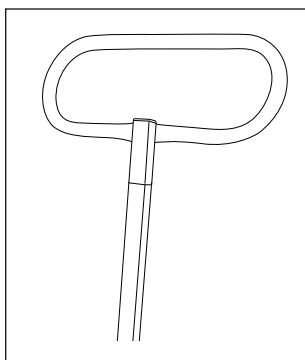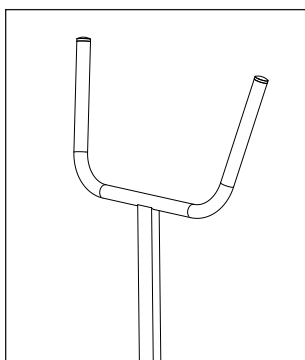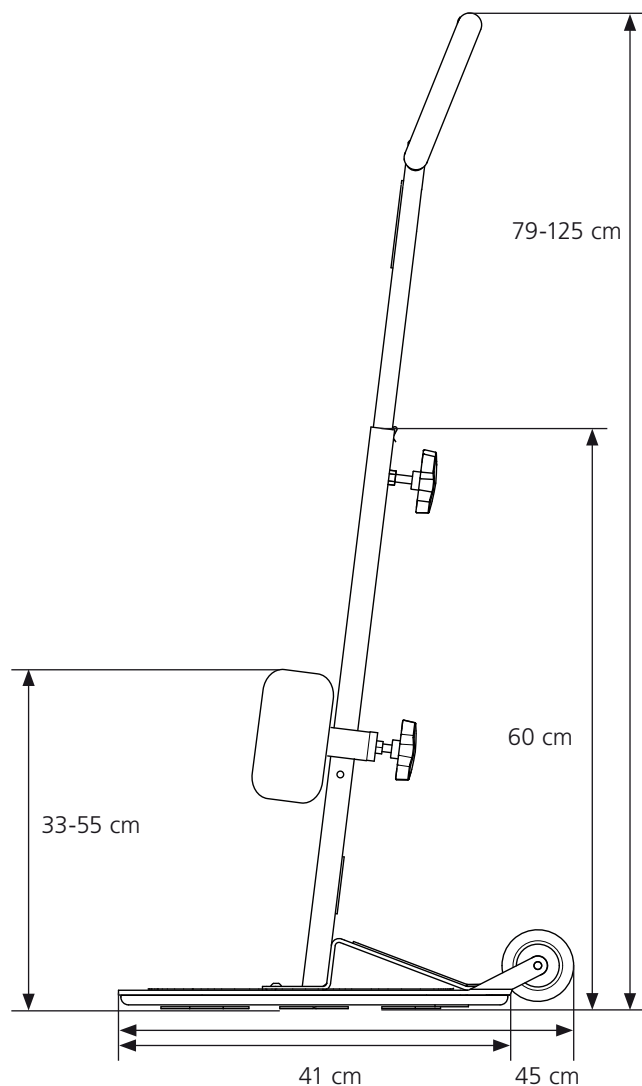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- EN**
1. Begin by positioning the new seat (chair, wheelchair, toilet or bed) in line with the existing seat.
 2. Slide Turner in under the user's feet.
 3. Adjust the height of the lower leg support so that its upper edge lines up just under the knee of the user.
 4. Adjust the height of the handle to a position that is comfortable for both user and carer.
 5. Grip the handle with both hands and place one foot on the pedal.
 6. The user grips the handle, if possible with both hands.
 7. Encourage the user to stand up while, at the same time, counterbalancing the user's weight using a foot and hands (use your bodyweight by leaning backwards).
 8. Twist Turner so that the user is transferred to the front of the new seat.
 9. Step on the pedal, brace your weight and ask the user to sit down. Maintain eye contact with the user during the entire transfer process.

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EN Safety

- Turner must not be used to transport people or be tipped while someone is standing on the rotating plate.

Guarantee:

5 year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

The expected life, during normal usage, is approximately 10 years. If your chair needs reconditioning or repair, use only original parts from Etac or components of similar quality.

Etac will not be held responsible for damage or injury caused by use of non-original parts.

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EN Maintenance

1 Wheels / Wheel axles: Check that the tyres, bearings and wheel axles are not worn, and that the wheels turn easily.

2 Base plate: Check that the base plate has no damage and that the anti-slip surfaces are not worn. Make sure the support bar is firmly anchored in the base plate. If the base plate is hard to rotate, wipe the dirt and grease between the sliding surfaces and lubricate again with Etac lubricant.

3 Leg Support: Check that the height adjustment works and is tightened. Make sure the pad does not have any damage.

4 Height controls: Check that the height controls work and are tightened.

5 Handle: Make sure the handle does not have any damage.

- Keep the wheels and the space between the rotating plates free of dust, hair and suchlike. If required, wipe off any grease between the sliding surfaces and lubricate with silicone grease.
- Check that the screws and controls are tightened properly. Perform inspection at least every 12 months.

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Etac Sverige AB
Etac AB (export)
Etac A/S