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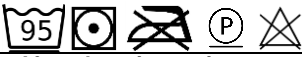
User manual/ instruction for use

Antidecubitusmattress / Decubitus therapy system

Please read the instruction and the notes on safety carefully before using the system to ensure safe operation

Product name	Hyper Foam 1 ** <i>Hyper Foam 1 Clinic</i>	Hyper Foam Plus ** <i>Hyper Foam Plus Clinic</i> <i>Hyper Foam Visco Clinic</i>	Hyper Foam 2 ** <i>Hyper Foam 2 Clinic</i>	Maxx250 ** <i>Maxx250 Clinic</i>
Scope of delivery	Mattress with foam core and PU-TEX Cover with zipper, ins truction for use			
Indication	Decubitusprophylaxis	Decubitus Therapy up to stage 3 (EPUAP) middle risk	Decubitus Therapy up to stage 4 (EPUAP) high risk	Decubitus Therapy up to stage 4 (EPUAP) high risk
Contraindikation	none			
Range of application	Decubitusprophylaxis	Decubitustherapy to grade 3 (EPUAP) middle risk	Decubitustherapy to grade 4 (EPUAP) high risk	Decubitustherapy to grade 4 (EPUAP) high risk
Patient weight min.-max.	to 90 kg	20 kg - 120 kg <i>20kg – 140kg</i>	20 kg - 120 kg <i>20kg – 150kg</i>	40 kg - 250 kg
Material	PUR- foam Cover PU-coating	PUR- foam Cover PU-coating	PUR- foam Cover PU-coating	PUR- foam Cover PU-coating
Sizes	width: 90-100 cm length: 190-220 cm			
Weight of product (90/200 cm)	9 kg	11 kg	12 kg	18 kg
Medical device class 1 according 93/42 EWG and medical product law (MPG)				
Technical safety/ flame retardancy	DIN EN 597 I + II // BS 7175 sec 3 crib 5			
Biocompatibility	DIN EN ISO 10993-5 ; DIN EN ISO 10993-1			
Notes on safety	Suitable for beds with adjustable lying surface side bars of the bed must be at least 22cm above surface mattress			
guaranty	3 years	3 years	5 years	5 years

Cleaning/ maintenance/ further use

Homecare	Clinical use
While using	
Cover disinfection by wiping	Cover disinfection by wiping
	
After changing patient	
Cover disinfection by wiping	Cover disinfection by wiping
Cover disinfection by wiping	autoclaving
	
No further maintenance necessary	

Instructions for storage and use

1. After receipt, the mattress must be taken out of the cardboard box immediately
2. Put the mattress onto the bed.
3. Please make sure that the arrows on the label („top side“ + mattress name) point upwards.

Manufacturer according to the MPG (German Devices Act)

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Funke Medical AG